



Product data

0-30° lateral positioning system for prevention and treatment of pressure ulcers. The Turn All® System turns a standard bed into an automated lateral turning bed.



Easy to set up, clean and use. Dial sets the timer for operation and the "On/off" button control air flow and alternating pressure feature. Cycles complete every 90 minutes, turning the patient from side to side 25 – 30 degrees, in each direction.

The system automatically turns the patient and is designed to gently and quietly turn the patient, without the intervention of a healthcare professional.

Product description

The new Levabo Turn All® lateral turning system is developed for acute and homecare patients.

The system is to support clinicians and carers in the regular turning of primary immobile patients, who are at the highest risk of developing pressure ulcers.

Several clinical practice guidelines recommend repositioning at least every two hours for immobile patients and elderly people. Also the 2019 International Pressure Ulcer guidelines (EPUAP, NPIAP, PPIA) recommend the use of the 30° lateral side-lying position.

Fits any standard bed where it is placed under the existing mattress. Also suitable on beds with a rising backrest.

Side Support System

Turn All® turning system is equipped with a support system for the opposite side of the primary lift.

The support system eliminates the feeling of sliding away from the primary lift side, and reduces the risk of shear.



Advantages:

- Comprehensive pressure relief
- Prevention of pressure injuries / ulcers
- Treatment of existing injuries / ulcers
- Comfortable and doesn't interrupt normal sleep
- Quiet when used
- Bariatric/heavy patients can be carefully and easily turned thereby

- protecting the health and wellbeing of staff working with them
- Full overview as the patient is always lying on their back on the mattress
- Gentle positioning even for patients in pain
- Slight lifting of the mattress, on the reverse side of the primary lift side,

- provides support and reassurance as well as decreases the risk of shear
- Time saving results in more care time available for the patient
- Improved microclimate and blood circulation
- Robust, durable and reliable
- Can be integrated into any care bed with removable mattress.

Turn All® product range

| Product name | Size in cm | Max weight in Kg | Product no. | Application |
|--|--------------|------------------|-------------|--|
| Turn All multi-patient 3C incl cover w/ TPU lower part | 85 x 200 | 225 | 70128 | System containing Turn All® lifting mattress, 1 Turn All® cover w/ TPU lower part, and 1 pump, 3 cycles: 30/30, 60/60, 90/90 minutes |
| Turn All Cover with TPU lower part | 85 x 200 | N/A | 70114 | Cover for the Turn All® lifting mattress w/ TPU lower part. |
| Turn All Pump 3C | 27 x 13 x 10 | N/A | 76117 | Pump for the Turn All® system, 3 cycles: 30/30, 60/60, 90/90 minutes |
| Bariatric | | | | |
| Turn All multi-patient 3C Bariatric with cover with TPU lower part | 110 x 210 | 400 | 70098 | System containing Turn All® lifting mattress, 1 Turn All® cover w/ TPU lower part, and 1 pump, 3 cycles: 30/30, 60/60, 90/90 minutes |
| Turn All Cover Bariatric with TPU lower part | 110 x 210 | NA | 70126 | Cover for the Turn All® Bariatric lifting mattress w/ TPU lower part. |